


# A4 | PROPOL therapy

## CLEANING & MAINTENANCE

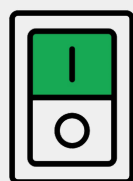

### SAFETY FIRST

Switch off the device before each cleaning operation and disconnect the mains cable from the socket.

### DIFFUSER CLEANING

Use a damp cloth with antibacterial cleanser for the device and exterior cleaning of the connecting tube.



### ACCESSORIES SANITIZING (MASK AND TUBE)

Possibility of various solutions:

- 1) In hot water (40° C) with mild dish detergent;
- 2) By immersion in a solution of water and vinegar, in the proportions of  $\frac{3}{4}$  and  $\frac{1}{4}$ , respectively, or in water and sodium bicarbonate;
- 3) Soak in cold water with a small glass of disinfectant for about 15 minutes.

### RINSING AND FINAL DRYING

After cleaning, the components must be drained well on a clean surface. Then dry with a soft cloth. Before storing the device, make sure it is completely dry.



### USE OF THE MASK

The mask is for personal use, therefore it can be used by a single patient, to get the maximum benefit and avoid the spread of germs and bacteria.

